

CUPID LIMITED, INDIA

WE HELP THE WORLD PLAY SAFE...

*Manufacturer and Supplier of Male Condoms,
Female Condoms & Water based Lubricants.*

TRAINING PROGRAM

CUPID FEMALE CONDOM

Website:- <http://www.cupidlimited.com>

Email:- info@cupidlimited.com

Address:- A-68, M.I.D.C. (Malegaon), Sinnar, Nashik - 422113,
Maharashtra, India.

Phone:- +91- 2551-230280

Social Media:- @cupidcondoms



CUPID[®]

WHAT IS CUPID FEMALE CONDOM?

- **Cupid Female condom** is a **pre-lubricated** with Silicone. Thin, soft, loose-fitting sheath made from **Natural Rubber Latex** which is worn **inside the vagina**.
- There is a flexible **octagonal ring** at open end and donut shaped soft polyurethane open cell foam at the closed end.
- The **polyurethane foam** at the closed end of the sheath is used to insert the condom inside the vagina and hold it in place during intercourse.
- The octagonal outer ring at the open end of the sheath remains outside the vagina and covers part of the external genitalia, it also helps to prevent condom being pushed inside during use.
- **Cupid female condom** is inserted into the vagina **before sex**. It holds the man's sperm after ejaculation thus helping **prevent unintended pregnancy**. It also acts as a **barrier to sexually transmitted infections (STI's), including HIV**.



THIS IS HOW THE CUPID FEMALE CONDOM LOOKS LIKE:




Red Colored, Strawberry Scented

External Retainer Ring

Sheath

Internal Retainer Sponge

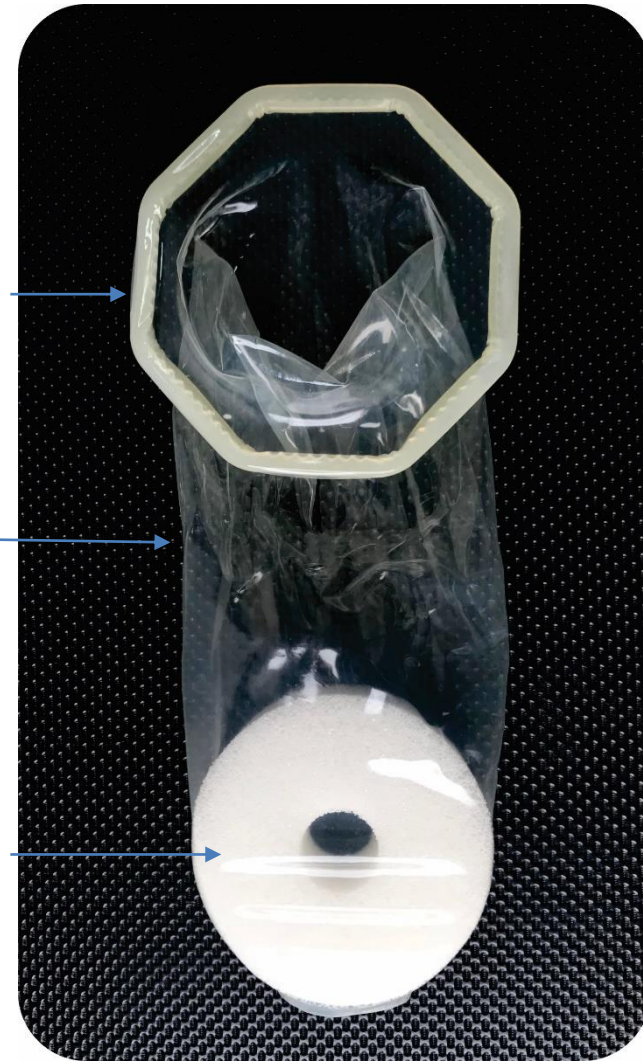

Natural Colored, with/without Vanilla Scented



WHO CAN USE CUPID FEMALE CONDOM?

- Women who want to **prevent** unintended pregnancy and protect from STI's / HIV.
- People who **do not want to use a male condom** or are looking for an alternative.
- Women who are **menopausal** (both Pre and Post).
- People who are **HIV positive**.
- Women who have recently **given birth**.
- Women who are **menstruating**.
- Women who **have** a retroverted uterus.

BENEFITS OF CUPID FEMALE CONDOM

- Correct and consistent use of **Cupid Latex Female Condoms** will help to **prevent** unwanted pregnancies and transmission of many Sexually Transmitted Diseases (STDs) including HIV/AIDS.
- **Cupid Female Condoms** can be worn by women well before the foreplay and vaginal intercourse.
- **Cupid Female Condom** does **not have any side effects** except for latex allergy on some individuals.
- **Cupid Female Condoms** are **Easy to Use and Unique** by its design.



HOW TO USE CUPID FEMALE CONDOM?

READ USAGE INSTRUCTIONS CAREFULLY AND FOLLOW:

Step 1: Open the female condom package carefully; tear at the notch on the top right of the package. Do not use scissor or a knife to open.

Step 2: The outer ring of the female condom covers the area around the opening of the vagina. The inner sponge is used for insertion and to help hold the sheath in place during intercourse.

Step 3: While holding the sheath at the closed end, grasp the sponge and squeeze it with the thumb and second finger so it becomes long and narrow.

Step 4: Choose a position that is comfortable for insertion; squat, raise on leg, sit or lie down. Gently insert the closed end of the product into vagina. Female Condom should be inserted into the vagina before any contact occurs between the vagina and the partner's body to assist in the prevention of STIs and pregnancy.

Step 5: Place the index finger on the inside of the condom and push the inner sponge up as far as it will go. Be sure the sheath is not twisted. The outer ring should remain on the outside of the vagina.



Step 6: The female condom is now in place and ready for use with your partner.

Step 7: Be sure that your partner's penis is not entering from the sides, between the sheath and vagina wall. If any of the partner feels the female condom is slipping into or out of the vagina, immediately stop and check.

Step 8: Gently guide your Partner's penis into the sheath's opening with your hand to make sure that it enters properly.

Step 9: Withdraw the penis soon after ejaculation leaving the female condom in place in the vagina. To remove the female condom, after the use, twist the outer ring and gently pull the condom out.

Step 10: Wrap the condom in the package or in tissue and throw it in the garbage. Do not put it into toilet.




CUPID®

HOW TO USE CUPID FEMALE CONDOM?

- Before opening the Cupid Female Condom pack, check the expiry date. Do not use expired condoms.
- Also check the pack for any damage, if the pack is torn or has any holes in it do not use the pack, throw it away and use a fresh pack.

HOW TO USE **CUPID FEMALE CONDOM?**

STEP: 1

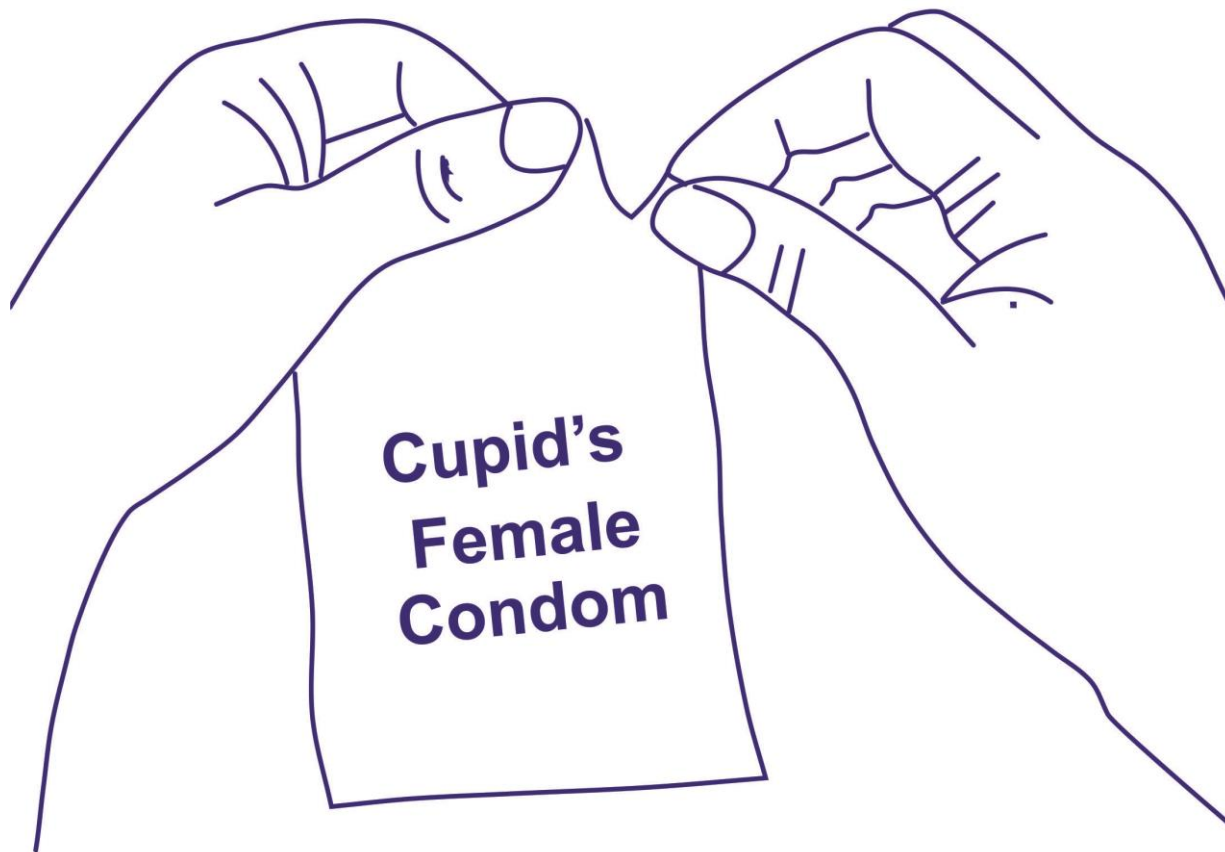


Open the female condom package carefully; tear at the notch on the top right of the package. Do not use scissor or a knife to open.



HOW TO USE CUPID FEMALE CONDOM?

STEP: 2

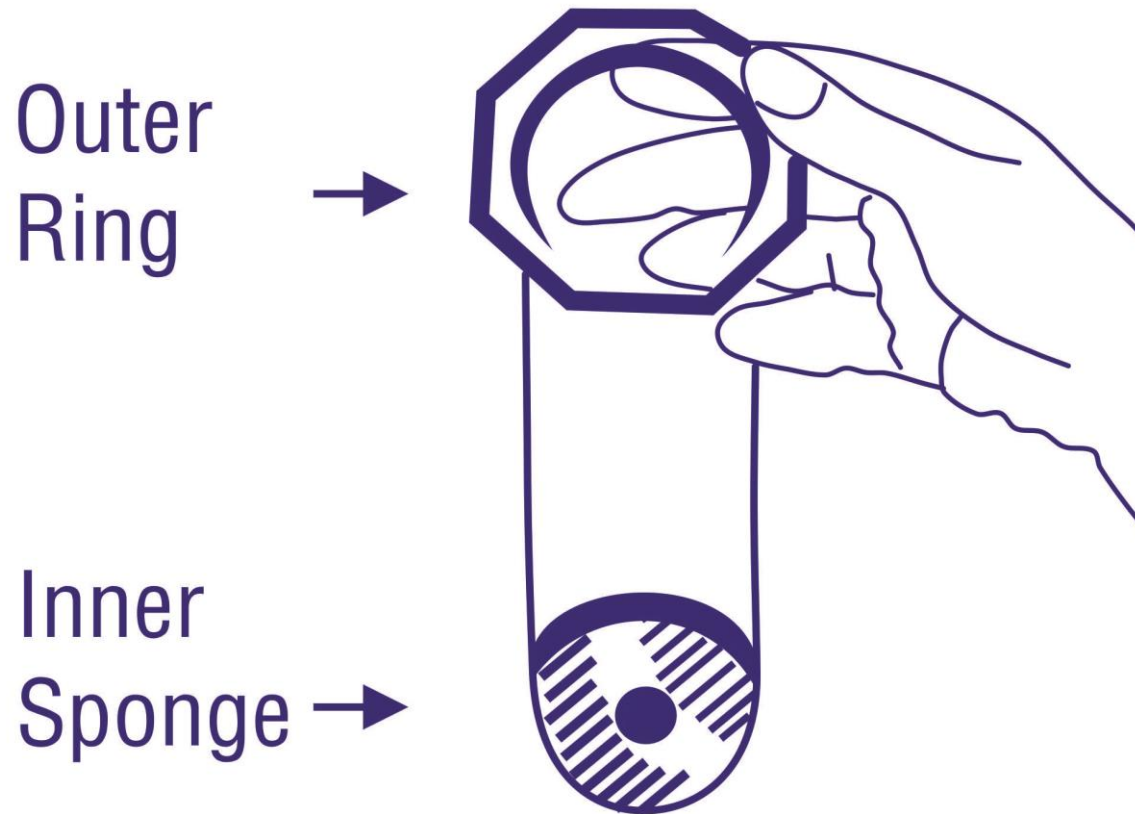

The outer ring of the female condom covers the area around the opening of the vagina. The inner sponge is used for insertion and to help hold the sheath in place during intercourse.

HOW TO USE CUPID FEMALE CONDOM?


STEP: 3



While holding the sheath at the closed end, grasp the sponge and squeeze it with the thumb and second finger so it becomes long and narrow.

HOW TO USE CUPID FEMALE CONDOM?

STEP: 4

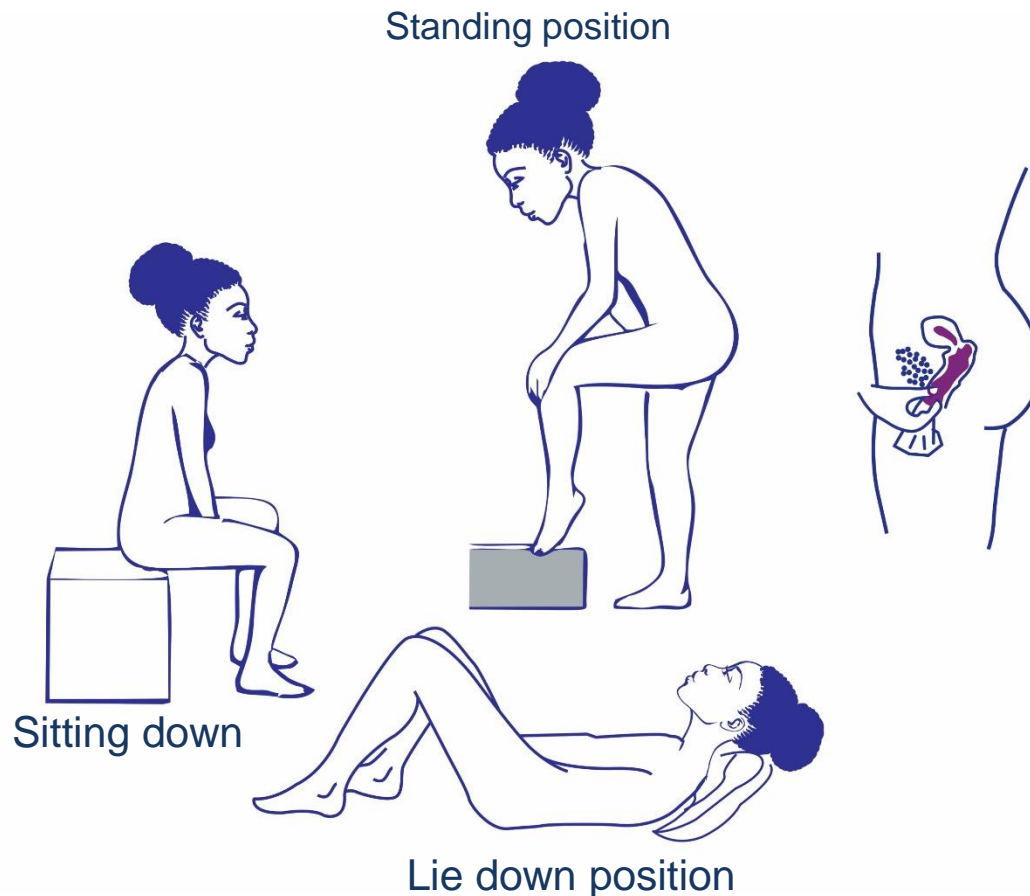

Choose a position that is comfortable for insertion; squat, raise one leg, sit or lie down. Gently insert the inner sponge into vagina. Female Condom should be inserted into the vagina before any contact occurs between the vagina and the partner's body to assist in the prevention of STIs and pregnancy.

HOW TO USE CUPID FEMALE CONDOM?


STEP: 5



Place the index finger on the inside of the condom and push the inner sponge up as far as it will go. Be sure the sheath is not twisted. The outer ring should remain on the outside of the vagina.

HOW TO USE CUPID FEMALE CONDOM?

STEP: 6

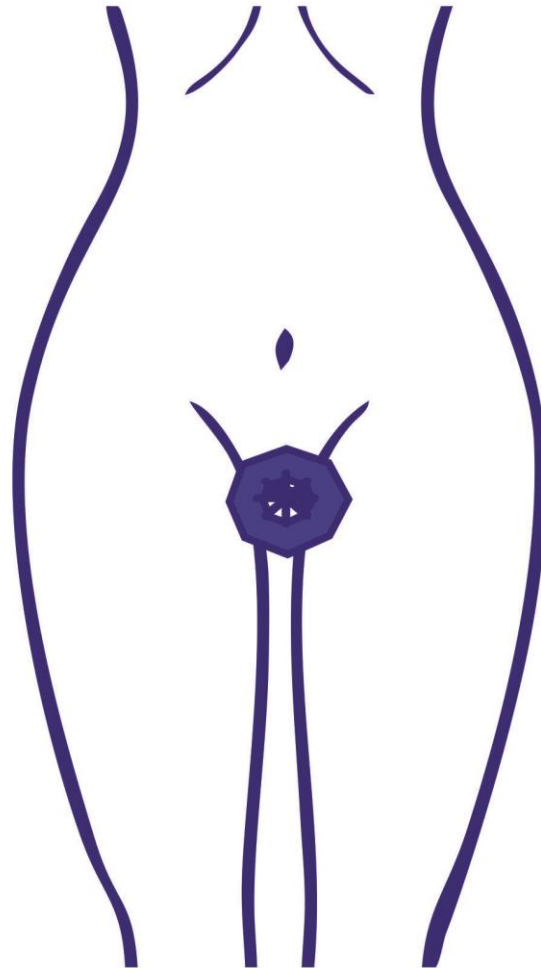


The female condom is now in place and ready for use with your partner.



HOW TO USE CUPID FEMALE CONDOM?

STEP: 7

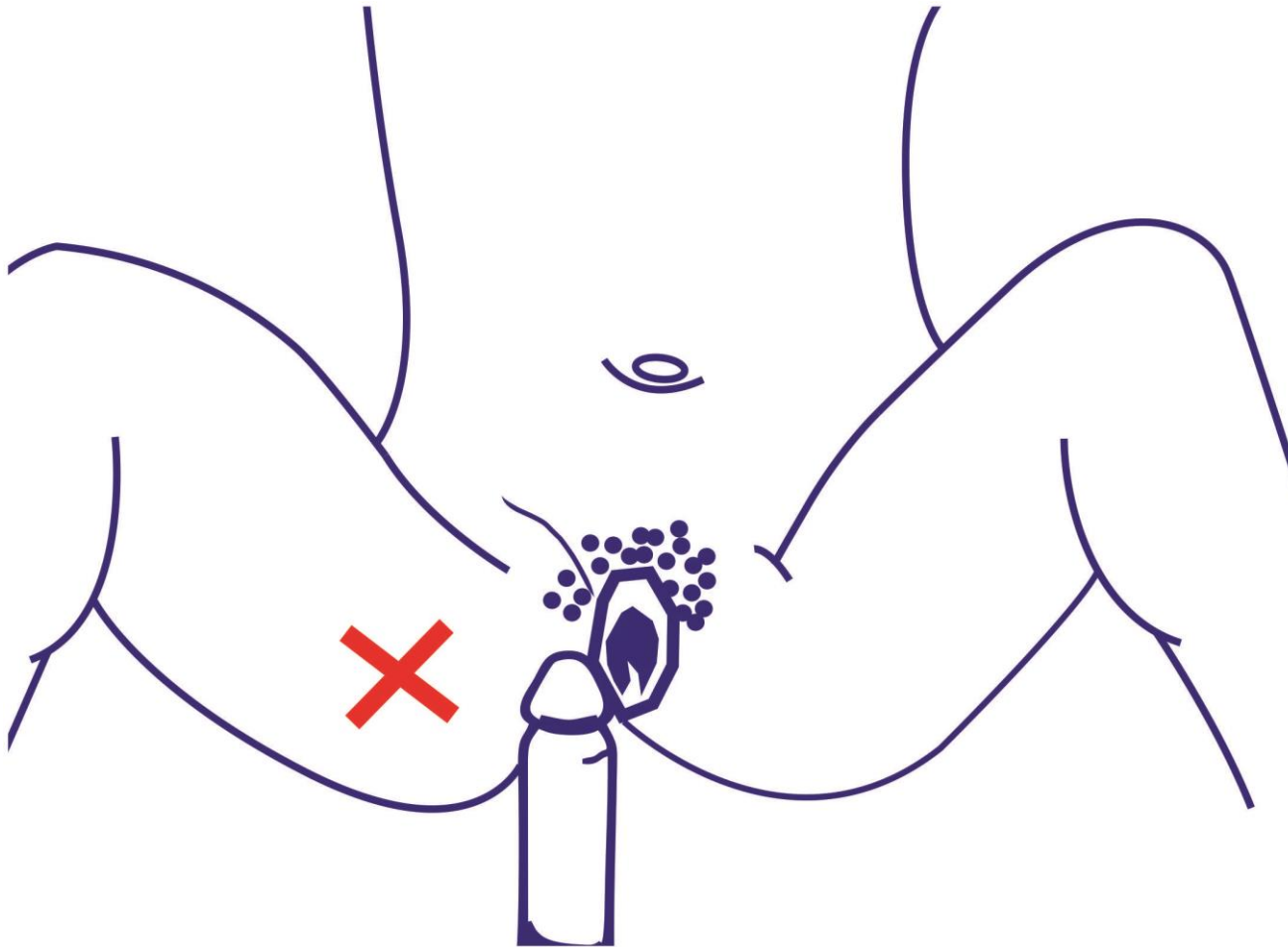

Be sure that your partner's penis is not entering from the sides, between the sheath and vagina wall. If any of the partners feels the female condom is slipping into or out of the vagina, immediately stop and check.

HOW TO USE CUPID FEMALE CONDOM?

STEP: 8



Gently Guide your Partner's penis into the sheath's opening with your hand to make sure that it enters properly.

HOW TO USE CUPID FEMALE CONDOM?


STEP: 9



Withdraw the penis soon after ejaculation leaving the female condom in place in the vagina. To remove the female condom, after the use, twist the outer ring and gently pull the condom out.

HOW TO USE CUPID FEMALE CONDOM?

STEP: 10

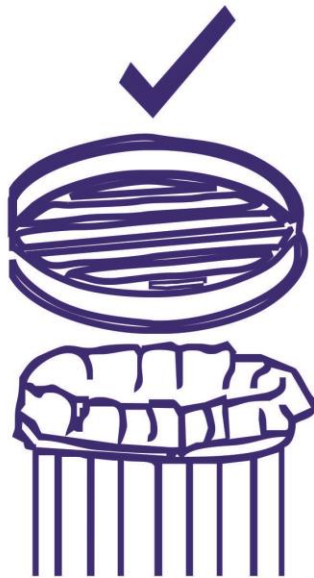

Wrap the condom in the package or in tissue and throw it in the garbage. Do not put it into toilet.

HOW TO USE CUPID FEMALE CONDOM?

You can also see the video on How to use Cupid Female condom in the below YouTube link

<https://www.youtube.com/watch?v=0hkVJiwvEl8&t=8s>



ADVANTAGES OF CUPID FEMALE CONDOM

- Cupid Female Condom is made up of Finest Quality Latex. Latex film is a proven barrier being used for billions of latex male condoms over decades.
- Cupid Female condoms are made with Polyurethane Foam as Internal Retainer as compared to rigid polyurethane ring in currently available female condom. The polyurethane open cell foam helps for Easy insertion, Better anchoring, Comfortable while use, Absorbs semen after ejaculation, Avoid spillage of semen while removal of condom after the use.
- Cupid Female condoms are fitted with Octagon shape ring at the open end that helps to properly hold the open end of the condom stable outside the vagina during the use and also covers the genitalia to protect from potential contact of bodily fluids.
- Cupid Female condoms are available in both Natural & Pink color with/without flavors. Colors and flavors are preferred by many women.
- Cupid Female condoms are easy to use and economical.



HOW IS **CUPID FEMALE CONDOM** DIFFERENT FROM OTHER FEMALE CONDOMS?

- **Cupid Female Condom** are made up of Natural rubber latex and hence economical when compared with female condoms made out of other materials.
- Since the inner retainer is made of soft sponge it may give extra pleasure to both the partners during intercourse.
- The sponge helps in absorbing the semen after ejaculation and does not let semen spill out while removing the condom.
- Easy to insert **Cupid Female Condom** as the sponge is easy to squeeze.
- It is well lubricated from both inside and outside.



CUPID®

USER'S FEEDBACK

- It is very easy to insert by squeezing the sponge – Female Partner, Regular Couple.
- Felt very good and got stimulated while insertion of cupid female condom with sponge – Male Partner, Regular couple.
- Lubrication is very good – Female Sex Worker.
- After insertion of condom, it is giving good feeling/pleasure and very comfortable due to the soft sponge, even before the sexual act – Female Partner, Regular couple.
- The act with this condom having sponge inside is more pleasurable without compromising safety – Male & Female Partner, Regular couple.
- Male Partner preferred Cupid Female Condom over male condom because it is well lubricated in both sides and undoubtedly stronger than male condoms – Regular Partner of Female Sex Worker.
- When the head penis touches the sponge inside, got stimulated during the act. She also felt the punch of pleasure – Male Partner, Regular Couple.
- My husband helped insertion; he really enjoyed and says, the sponge is giving fantastic feeling – Female Partner, Regular Couple.
- I wore this female condom half-an-hour before fore-play and enjoyed uninterrupted act – Female Partner, Regular couple.
- Color and Flavor was really very nice. We both enjoyed. I use it for preventing pregnancy, because it does not have any side effects. Best Option – Female Partner, Regular Couple.
- The material is elastic, strong and no chance to tear. Because there is nothing catches the penis, I enjoyed – Male Partner, Regular Couple.



USER'S FEEDBACK

- I prefer this Cupid Female condom because, it has no side effects. It does not diminish the pleasure of the act- Female Partner, Regular Couple.
- There is no feeling of something in-between; on the contrary it was pleasurable – Male Partner, Regular couple.
- Because it is stronger, mentally free during the act and enjoyed –Regular Partner of Female Sex Worker.
- Best part of this Cupid Female Condom is, the sponge absorbs semen and never allows to spill while removing and disposing – Female Partner, Regular couple.
- I will prefer to continue using this Cupid Female Condom regularly if made available – Female Sex Worker.
- I will definitely prefer to use this condom next time – Female Partner, Regular Couple.
- My customers prefer me using this Female Condom with sponge and even ready to pay me more – Female Sex Worker.
- Able to attract more customers, including those who otherwise do not wish to use male condoms. Moreover these customers liked Cupid Female Condom.
- Different Experience, Good, Pleasurable, Extra Safety & Satisfied – Health Worker sharing her experience after using Cupid Female Condom.



CUPID®

CUPID FEMALE CONDOM CREDENTIALS

Cupid Female condom has the following certifications and accreditation:-

- ISO 9001:2008
 - ISO 13485:2003
 - ISO 14001:2004
 - ISO 25841:2017
 - WHO GMP
 - CE 2460
 - SABS CERTIFIED
 - UNFPA PREQUALIFIED
- Cupid Female Condom are exported to more than 40 countries worldwide.

CAUTIONS FOR USERS OF CUPID FEMALE CONDOMS:

- Store in cool dry place away from heat and direct sunlight.
- Do not use the condom after expiry date indicated on the package.
- **Cupid Female Condoms** are very easy to use. But may need practice for insertion.
- Trim the nail of the finger used for insertion of female condom to avoid tear of condom while insertion.
- **Cupid Female Condoms** contains Natural Rubber Latex which may cause allergic reaction to some individuals.
- Do not use **Cupid Female Condoms** with oil based lubricants. If necessary use water based lubricants.
- Do not use Male and Female Condoms together.
- Any use of **Cupid Female Condoms** for other than vaginal intercourse may increase the potential of damage to the condom.
- Do not put the used **Cupid Female Condom** into the toilet. Wrap the condom in the package and throw it in the garbage.



CUPID®

FAQ (FREQUENTLY ASKED QUESTIONS) ABOUT CUPID FEMALE CONDOM

Q: Can I use the Cupid female condom during pregnancy?

A: You can use the Cupid female condom when you are pregnant since the condom doesn't move beyond the cervix. You can also use Cupid female condom after recently giving birth.

Q: Can Cupid female condom and a male condom be used together?

A: No, as the friction caused by the two condoms can lead to tearing of either one or both the condoms. Also, using one condom (male or female) provides equal protection to both partners.

Q: Why does the Cupid female condom seem so large?

A: The Cupid female condom is similar in length to a male condom. However, it is wider than a male condom as, once inserted, it lines the walls of the vagina and allows for movement of the penis inside the sheath. It also provides extra protection by covering part of the external female genitalia and the base of the penis.



CUPID®

FAQ (FREQUENTLY ASKED QUESTIONS) ABOUT CUPID FEMALE CONDOM

Q: When can I insert the Cupid female condom?

A: The Cupid female condom can be inserted in advance of sexual intercourse and does not need to be taken out immediately after intercourse.

Q: Can I reuse the Cupid female condom?

A: No, reuse of any condom is not recommended – male or female. A new condom should be used every time you have intercourse.

Q: Can Cupid female condoms disappear inside my body?

A: The Cupid female condom covers the cervix. The diameter of the opening to the cervix is 1 mm to 3 mm so it is impossible for Cupid female condom to disappear inside your body.

Q: Can I use the Cupid female condom during Menstruation?

A: The Cupid female condom does not interfere with menstruation and can therefore be used. However, the Cupid female condom should be inserted just before intercourse and removed soon after.



CUPID®

ABOUT CUPID LIMITED

- ✓ **Cupid Limited** is a leading manufacturer & supplier of Male Condoms, Female condoms and Water based lubricant jelly based in India.
- ✓ **Cupid Limited** is the first company in the world to have been Prequalified by WHO/UNFPA for supply of both male and Female condoms.
- ✓ **Cupid Limited** has a vast experience of more than 20 years in manufacturing and supply of high quality condoms.
- ✓ **Cupid Limited** has exported condoms to more than 40 countries worldwide.
- ✓ **Cupid Limited** has its own in-house Research & Development center.



FOR TRAINERS

Cupid Limited can provide **Demo Models** for trainers



CUPID LIMITED, INDIA

WE HELP THE WORLD PLAY SAFE...

*Manufacturer and Supplier of Male Condoms,
Female Condoms & Water based Lubricants.*

THANK YOU FOR ATTENDING THIS
TRAINING PROGRAM

Website:- <http://www.cupidlimited.com>

Email:- info@cupidlimited.com

Address:- A-68, M.I.D.C. (Malegaon), Sinnar, Nashik - 422113,
Maharashtra, India

Social Media:- @cupidcondoms



CUPID[®]