



CUPID LIMITED

Manufacturer and Suppliers of Male Condom, Female Condom & Lubricants



CUPID - Over 20 years of expertise, trust and reputation

- Cupid Limited has been in manufacturing and sale of condoms since 1998.
- Cupid Limited is the only company in the world to obtain pre-qualification status from WHO/UNFPA for supply of both Male & Female Condoms.

CUPID FEMALE CONDOM:


CUPID Female condom is thin, soft, loose-fitting sheath made from Natural Rubber Latex which is worn inside the vagina. There is a flexible Octagonal ring at open end and donut shaped soft polyurethane foam at the closed end. The polyurethane foam at the closed end of the sheath is used to insert the condom inside the vagina and hold it in place during intercourse. The octagonal outer ring at the open end of the sheath remains outside the vagina and covers part of the external genitalia. CUPID Female Condom is inserted into the vagina before sex. It holds the man's sperm after ejaculation thus helping prevent unintended pregnancy. It also acts as a barrier to sexually transmitted infections (STIs), including HIV.

FEMALE CONDOM - A New Prevention Tool :

- Can increase a woman's sense of empowerment as she is taking the initiative to protect herself and her partner.
- Men do not have to have / Maintain an erection to use Female Condom. Female Condom is a good option for men who are unable to maintain an erection while using male condom.
- Men do not have to withdraw right after ejaculation, which may lead to more intimacy when using Female Condom.
- Female Condom can be inserted ahead of time so there are no interruptions to "spoil the mood".
- Insertion of Female Condom can be incorporated into sexual play, the partner can watch as it is put in place OR can help insert it.
- Female Condom provides an option when couple does not want to use the male condom.
- Female Condom is not tight or constricting around the penis; some men prefer Female Condom to the male condom.
- External ring rubs against clitoris and increases pleasure for many women.
- The Internal Retainer of Female Condom can stimulate the tip of the penis during intercourse.
- The man does not have to worry about wearing a condom.
- Female Condom provides effective dual protection, reducing the worry during sex about pregnancy and STI's / HIV.

Cupid[®] female Condom

PRODUCT DESCRIPTION

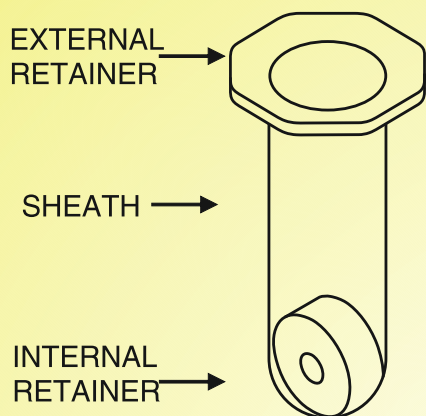

ADVANTAGES OF CUPID FEMALE CONDOM:


- Meets & Exceeds requirement of ISO 25841/SANS 25841:2011.
- Unique by its design.
 - Only Female Condom available in Various Colors.
 - Only Female Condom available with Vanilla Scent.
 - Meets WHO/UNFPA Generic specifications for Female Condoms-2012.
 - Economical / Cost Effective & Easy to use.
 - Made from finest Quality Latex.

ACCREDITATIONS / CREDENTIALS:


- ISO 9001:2008
- ISO 13485:2003
- ISO 14001:2004
- WHO-GMP
- CE-2460
- SABS MARK
- WHO / UNFPA Pre-qualified

USAGE INSTRUCTIONS


Step 1: Open the female condom package carefully; tear at notch on the top right of the package. Do not use scissor or knife to open.

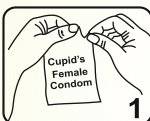

Step 2: The outer ring of the female condom covers the area around the opening of the vagina. The inner sponge is used for insertion and to help hold the sheath in place during intercourse.

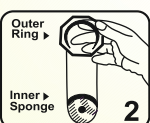


Step 3: While holding the sheath at the closed end, grasp the sponge and squeeze it with the thumb and second finger so it becomes long and narrow.

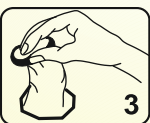

Step 4: Choose a position that is comfortable for insertion; squat, raise one leg, sit or lie down. Gently insert the inner sponge into vagina.



Step 5: Place the index finger on the inside of the condom, and push the inner sponge up as far as it will go. Be sure the sheath is not twisted. The outer ring should remain on the outside of the vagina.

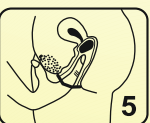

Step 6: The female condom is now in place and ready for use with your partner.



Step 7: Be sure that your partner's penis is not entering from the sides, between the sheath and vagina wall.



Step 8: Gently guide your partner's penis into the sheath's opening with your hand to make sure it enters properly.



Step 9: To remove the female condom, after the use, twist the outer ring and gently pull the condom out.



Step 10: Wrap the condom in the package or in tissue, and throw it in the garbage. Do not put it into the toilet.



Contact Details:

CUPID LIMITED

E-mail: ecommerce@cupidlimited.com Website: www.cupidlimited.com

Tel: +91 2551 230280 / 230772 / +91 7722009580